
# solventum

## Rotating Capsule Mixing Device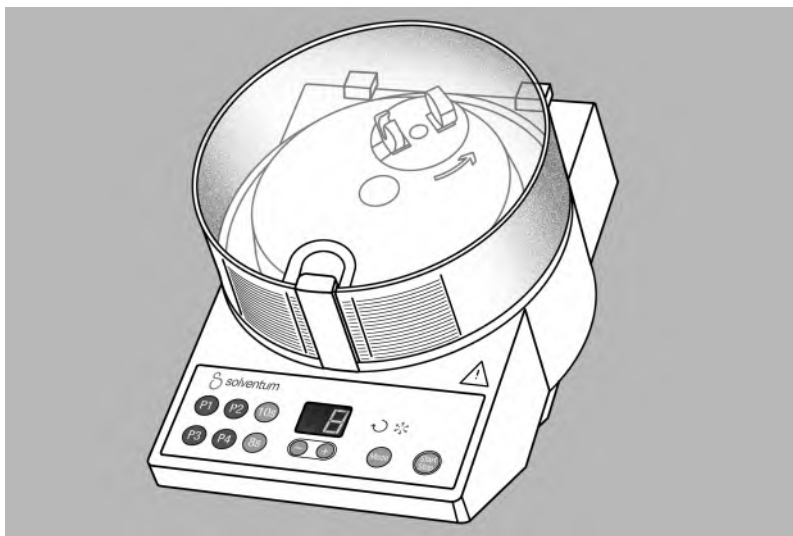

- (en) Rotating Capsule Mixing Device
- (de) Rotations-Kapselmischgerät
- (fr) Appareil de mélange à rotation pour matériaux dentaires en capsule
- (it) Miscelatore rotativo per materiali dentali in capsula
- (es) Aparato mezclador rotativo para materiales dentales en cápsulas
- (pt) Misturador rotativo para cápsulas
- (nl) Roterend mengapparaat voor capsules
- (el) Συσκευή περιστροφικής ανάμιξης για κάψουλες
- (sv) Rotationskapselblandare
- (fi) Pyrörökapselinsekoitin
- (da) Rotationsblendeapparat til dentale materialer i kapsler
- (no) Rotasjonsblendeapparat for kapsler

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## Rotating Capsule Mixing Device

Table of Contents	Page
1. Safety	1
2. Product Description	2
3. Fields of Application	2
4. Technical Data	2
5. Installation	3
5.1 Setting the Operating Mode	3
5.2 Setting the Mixing Time	3
5.3 Changing the Basic Settings	3
6. Operation	3
6.1 Inserting the Capsule	3
6.2 Mixing	4
6.3 Removal of the Capsule	4
7. Errors	4
8. Cleaning and Disinfection	5
9. Return of Old Electric and Electronic Equipment for Disposal	5
9.1 Collection	5
9.2 Return and Collection Systems	5
10. Customer Information	5
Symbol Glossary	6

### 1. Safety

#### CAUTION!

Please read these instructions carefully before you set up and start the device. As with all technical equipment, this unit will function properly and be safe to operate only if both the general safety regulations and the specific safety instructions laid out in these Operating Instructions are followed.


- The unit must be operated in accordance with the following instructions. We accept no liability for any damage arising from use of the unit for any other purpose.
- Prior to setting-up the device, ensure that the voltage stated on the rating plate is identical to the available main voltage. Use of the device at different voltages may lead to the destruction of the unit.
- The unit must be operated with the cover intact and closed, otherwise, there is a risk of injury.
- The cover must not be opened until the capsule holder has come to a complete stop.
- Only the capsule to be mixed but no other objects may be present in the space below the cover prior to initiating the mixing cycle.
- Placement or operation of the unit on non-level surfaces is not permissible. Set up the Rotating Capsule Mixing Device on a dry, non-slip surface.
- Transferring the unit from a low to a higher ambient temperature, there may be a hazard due to condensation of water on the device. Hence, the unit should only be operated once it is fully equilibrated with the ambient temperature.
- For operation of the unit, connect the main plug to an adequate electrical socket with grounded lead. If an extension cable is used, ensure that the grounded lead is not interrupted.
- Maintain a safety area of 300 mm around the device. Do not stay within the safety area any longer than necessary for operation. Do not place any potentially hazardous substances in the safety area.
- Do not lean on the device during operation.
- Check the cover and the rotor assembly for damage at regular intervals. If a defect is found, stop operation of the device.
- If there is any reason to believe that the safety of the device may be compromised the unit must be taken out of operation and labelled appropriately to ensure the device is not inadvertently used by another person. The safety may be compromised, for example, when the unit fails to function properly or shows visible signs of damage.
- Only personnel authorized by Solventum Germany GmbH may open the unit housing and repair the device.
- Only adequately labelled original Solventum replacement parts may be used in the repair and maintenance of this unit. Any damage resulting from the use

of third-party replacement parts voids the liability of Solventum Germany GmbH.

15. Do not attempt to operate this unit by bypassing the protective mechanisms (e.g. by turning off the automatic shut-off switch on the cover of the unit). Violation of the safety regulations may cause serious injury and voids all warranties and guarantees.
16. Keep solvents and sources of intense heat away from the unit as they may damage the plastic housing.
17. The use of flammable or explosive materials in the device is prohibited.
18. The use of materials in the device, that may cause a hazard by a heavy chemical reaction with each other, is prohibited.
19. Cleaning and disinfecting the unit, ensure that no cleaning or disinfecting agent enters the device since this may lead to electric short circuiting or hazardous malfunctions.
20. Pay attention to capsule leak tightness when mixing. Capsules that are not leak tight contaminate the device with mercury. Contaminated devices cannot be repaired and must be disposed of as contaminated waste according to local laws. See the safety data sheet for the amalgam used for first aid, measures to take in case of unintended release, handling, storage, exposure limitation, and personal protective clothing.

## 2. Product Description

The Solventum™ Rotating Capsule Mixing Device unit is a mixing device for dental filling materials and cements in capsules. Unlike conventional capsule mixers, the device uses a rotating motion which is faster and more efficient and leads to improved homogeneity of the mixed pastes.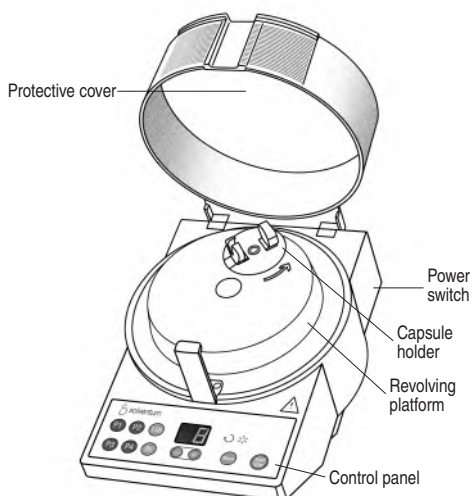

In addition, Rotating Capsule Mixing Device offers the option to centrifuge the capsules after mixing. Depending on the viscosity this procedure may reduce air bubbles in the material. Desired mixing and centrifugation times may be programmed into the mixing programmes "P1"- "P4". Moreover, two pre-set mixing times, "10s" and "8s", can be selected.

The device requires no maintenance.

Rotating Capsule Mixing Device is a multiple patient multiple use device.

These Instructions for Use should be kept for the duration of product use. The product may only be used when the product labeling is clearly readable. For details on all additionally mentioned products please refer to the corresponding Instructions for Use. The Instructions for Use are available at: <https://eIFU.Solventum.com>.

### Intended Use

Mixing device for dental filling and luting materials in capsules.

## 3. Fields of Application

Dental materials packaged in capsules with a max. capsule weight of 6 g.

Capsules with a weight of more than 6 g must not be used. Capsule materials of third-party manufacturers (including amalgam capsules) may be mixed with the Rotating Capsule Mixing Device provided they are suitable for the device in terms of weight and dimensions (please refer to the section, Technical Data). The optimal mixing time for mixing with the Rotating Capsule Mixing Device should be inquired from the capsule manufacturer.

### Intended Patient Population

All patients requiring dental treatment, unless the patient's condition limits the use.

### Contraindications

None known.

### Intended Users

Educated dental professionals who have theoretical and practical knowledge on usage of dental products.

### Potential Side-Effects

None known.

### Residual Risks

In situations with improper in-office infection control, or if the Instructions for Use are not followed, the following risks may occur: Carpedal spasm, cardiac arrhythmia, infection, laceration or other wounds.

## 4. Technical Data

Drive unit:	Ball-bearing motor with integrated self-resetting overheat cut-off
Operating mode:	Short-term operation, 2 min On/ 5 min Pause (to prevent needless motor heating)

Controller:	Microprocessor-controlled, high-precision mixing time control by quartz timing mechanism
Capsule holder:	Self-locking
Capsules:	Diameter max. 19 mm, total length max. 58 mm, length without tubes max. 38 mm, weight max. 6 g
Housing:	Housing and protective cover consist of high impact-resistant plastics
Mixing speed:	typically 2850 for 6 g-capsules
Centrifugation speed:	typically 2950 for 6 g-capsules
Operating temperature:	18°C...40°C/64°F...104°F
Humidity:	max. 80% at 31°C/88°F, max. 50% at 40°C/104°F
Main voltage:	120 V, 230 V, 50/60 Hz, for voltage setting refer to the rating plate
Power requirements:	390 VA
Fuses:	230 V: 2 A time-lag fuse 120 V: 3.2 A time-lag fuse
Dimensions:	(W x H x D) 180 x 210 x 250 mm
Weight:	4000 g
Continuous sound pressure level:	= 73.8 dB(A)
Year of manufacture:	see rating plate

## 5. Installation

Prior to the first start-up remove the transport safeguard (foam ring) and instruction card.

Connect the Rotating Capsule Mixing Device unit to the power supply and switch on the unit with the power switch on the back of the device. The display shows the mixing time of programme 1.

The following basic settings are factory-set:

Programme 1 (key "P1"):	12 sec mixing, 3 sec centrifugation
Programme 2 (key "P2"):	9 sec mixing, 3 sec centrifugation
Programme 3 (key "P3"):	7 sec mixing, 3 sec centrifugation
Programme 4 (key "P4"):	5 sec mixing, 3 sec centrifugation
Fixed time (key "10s"):	10 sec mixing, 3 sec centrifugation
Fixed time (key "8s"):	8 sec mixing, 3 sec centrifugation

## 5.1 Setting the operating mode

With the key "Mode" the user may toggle between the two available operating mode options. If only the "↻" symbol is illuminated, the device mixes without centrifugation. If both the "↻" and "⌘" symbols are illuminated, the device mixes and then centrifuges the sample for 3 sec.

## 5.2 Setting the mixing time

Mixing times in the range of 1 sec to 39 sec may be selected. By pressing the keys, "+" or "-", the device increases or decreases, respectively, the mixing time by 1 sec. Depressing and holding the key, the device continues to count up or down.

## 5.3 Changing the basic settings

### P1 – P4

Set both the mixing time and operating mode as described above, then depress one of the programme buttons, "P1"–"P4", until two short signals are issued. This confirms that the desired settings have been stored in the selected programme.


### 10s / 8s

The two fixed mixing times of "10s" and "8s" can be programmed to be used both with and without centrifugation. As described above, select an operating mode with or without centrifugation, and depress and hold the desired fixed time button until two short signals are issued.


## 6. Operation

### 6.1 Inserting the capsule

**The opening mechanism of the capsule holder works only when the unit is turned on.**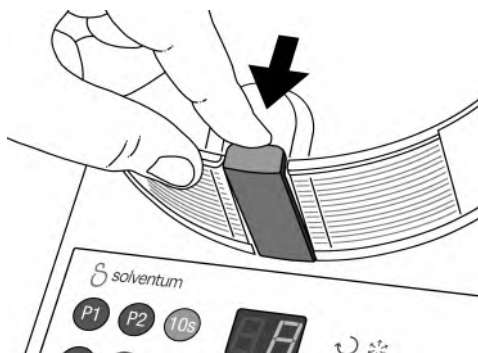

To open the cover, pull the locking lever forward in the direction of the arrow and pull the unlocked cover up.

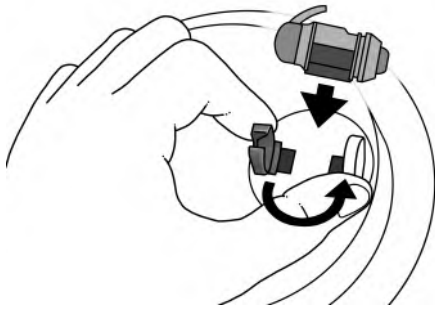Open the capsule holder with a counter-clockwise rotation. Place the activated capsule with the release opening in the blue half of the capsule holder, then release the capsule holder. The holder closes automatically.

**Do not operate the device with no capsule inserted!**

Do not use capsules weighing more than 6 g (Error message “Lo” is displayed).

## 6.2 Mixing

After closing the protective cover press the “Start/Stop”-button. This starts-up the mixing process. The mixing time and centrifugation time, if any, are counted down in the time display.

After the mixing time has elapsed, the latest time setting is re-displayed so that there is no need to re-enter any value if the same time is to be used.

The selected programme can be interrupted at any time by pressing the “Start/Stop”-button, upon which the program returns to its original setting.

**CAUTION:** For safety reasons, it is not feasible to start the unit unless the cover is closed. Attempting to operate the unit with the cover being open causes the time display to flash and an audible error message to be issued. Opening of the protective cover during mixing is associated with a high risk of injury. Hence, the mixing process is aborted and the error message “Lo” displayed. This may render the capsule useless! Therefore, never open the cover until the mixing process is completed and the capsule holder has come to a complete stop!

## 6.3 Removal of the capsule

Open the capsule holder with a counter-clockwise rotation, and remove the capsule.

## 7. Errors

Error	Possible cause	Remedy
Time display is off even though the unit is switched on	Power failure	Check both the power cable and main voltage
Motor fails to run when the unit is switched on	Cover of the unit is open Motor is overheated	Close protective cover Allow the unit to cool
Capsule is thrown out of the holder	Capsule holder is contaminated Capsule is too large or heavy (refer to Technical Data section)	Blast capsule holder with compressed air Use suitable capsules only
Error message “Lo” and error signal are issued	Cover was opened during mixing Motor or bearing are defective Capsule is too heavy	Press “Start/Stop” to quit Press “Start/Stop” to quit. For a short time, the speed (in revs per sec) at which the device was turned off is shown in the display. Have the unit repaired. Press “Start/Stop” to quit

## 8. Cleaning and Disinfection

### Caution

Make sure that the cleaning and disinfection agents you have chosen do not contain any of the following materials:

- Oxidizing agents (e.g., hydrogen peroxides)
- Oils
- Glutaraldehydes

Use only validated methods for cleaning and disinfection. Use solely cleaning and disinfection agents tested for efficacy and compatibility with the medical device (e.g., FDA certification). CaviWipes™ have been tested for use as a cleaning and disinfection agent. Always observe all applicable legal and hygiene regulations for dental offices and/or hospitals.

### General

The following procedure should be applied after use or before any subsequent use. The device must first be cleaned manually and, after cleaning, be manually disinfected in accordance with the following instructions.

### Manual Cleaning by Wiping

- ▶ Clean the device after every application with ready-to-use disinfectant wipes (e.g., CaviWipes™ based on the active substances alcohols and quaternary compounds) or equivalent cleaning agents. Follow the cleaning agent manufacturer's directions for use.
- ▶ Thoroughly clean the material surface of the device with two ready-to-use CaviWipes™ for at least 30 seconds or until no more dirt is visible.
- ▶ Special care must be exercised in disinfecting the grooves and connecting points.

### Manual Disinfection by Wiping (Intermediate Level Disinfection)

- ▶ Disinfect the device with ready-to-use disinfectant wipes (e.g., CaviWipes™ based on the active substances alcohols and quaternary compounds) or equivalent disinfection agents. Follow the cleaning agent manufacturer's directions for use, observing in particular the contact times.
- ▶ Disinfect the surface of the device for three minutes using at least two new disinfectant wipes. Keep all surfaces moist during the three-minute period.
- ▶ Special care must be exercised in disinfecting the grooves and connecting points.

### Inspection, Maintenance, Testing

- ▶ Examine the device for damage, discoloration, and contamination before every use.
- ▶ Do not use damaged devices under any circumstances. If the device is not visibly clean, repeat the cleaning and disinfection procedure.

## 9. Return of Old Electric and Electronic Equipment for Disposal

### 9.1 Collection

Users of electric and electronic equipment are required to collect their old equipment separately from other waste in accordance with the regulations of the specific country. Old electric and electronic equipment must not be disposed of with unsorted household waste. This separate collection is a prerequisite for recycling and reprocessing as an important method for preserving environmental resources.

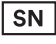
### 9.2 Return and Collection Systems

When your Rotating Capsule Mixing Device is no longer usable, do not dispose of the device with household waste.

## 10. Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet. Please report an undesirable outcome (e.g. adverse event, serious incident (EU)) occurring in relation to the device to Solventum and the local competent authority (EU) or local regulatory authority.

## Symbol Glossary

Reference Number and Symbol Title	Symbol	Description of Symbol
ISO 15223-1 5.1.1 Manufacturer		Indicates the medical device manufacturer.
Swiss Authorized Representative		Indicates the authorised representative in Switzerland.
ISO 15223-1 5.1.3 Date of Manufacture		Indicates the date when the medical device was manufactured.
ISO 15223-1 5.1.5 Batch code		Indicates the manufacturer's batch code so that the batch or lot can be identified.
ISO 15223-1 5.1.6 Catalogue number		Indicates the manufacturer's catalogue number so that the medical device can be identified.
ISO 15223-1 5.1.7 Serial number		Indicates the manufacturer's serial number so that a specific medical device can be identified.
ISO 15223-1 5.4.3 Consult instructions for use or consult electronic instructions for use		Indicates the need for the user to consult the instructions for use.
CE Mark		Indicates conformity to all applicable European Union Medical Device Regulations and Directives.
ISO 15223-1 5.7.7 Medical Device		Indicates the item is a medical device.
ISO 15223-1 5.7.10 Unique device identifier		Indicates a carrier that contains unique device identifier information.
Rx Only		Indicates that U.S. Federal Law restricts this device to sale by or on the order of a dental professional. 21 Code of Federal Regulations (CFR) sec. 801.109(b)(1)
Corrugated fiberboard		Indicates product packaging is made of corrugated fiberboard. Official Journal of the EC; Commission Decision (97/129/EC)

Reference Number and Symbol Title	Symbol	Description of Symbol
Low-density polyethylene		Indicates product packaging is made of low-density polyethylene. Official Journal of the EC; Commission Decision (97/129/EC)
Recycle electronic equipment		DO NOT throw this unit into a municipal trash bin when this unit has reached the end of its lifetime. Please recycle.
ISO 15223-1 5.1.8 Importer		Indicates the entity importing the medical device into the locale.

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© Solventum 2025. Solventum and the S logo are trademarks of Solventum or its affiliates.
U.S.A. 1-800-634-2249.