**DE** Vector® Fluid polish

**EN** Vector® Fluid polish

**FR** Vector® Fluid polish

**IT** Vector® Fluid polish

**ES** Vector® Fluid polish

## DE Produktinformation

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## EN Product information

**Vector® Fluid polish** is a medical device.

This product is only intended for professional dental applications together with the Vector® System of Dürr Dental SE and must only be used by trained specialists. The formulation is free of animal ingredients, gluten, lactose, fructose and microplastics.

### Indication:

Vector® Fluid polish is a suspension for use in the Vector® System, that gently and intensively cleans tooth, denture, and root surfaces with ultrasound dynamics. Due to this minimally invasive treatment with ultrasound dynamics, gingival pockets in particular are thoroughly flushed and the non-adhering plaque they contain is removed. Biofilms and subgingival calculus are also gently removed. Bacteria and endotoxins are reduced. Vector® Fluid polish is suitable for children and adults. Do not swallow.

### Contraindication:

Vector® Fluid polish is contraindicated in patients with known allergies to ingredients of Vector® Fluid polish (e.g. limonene).

### Composition:

Vector® Fluid polish contains hydroxyapatite, calcium phosphate, dispersing agents, PHB ester, adjuvants and aroma (contains limonene).

### Application:

Vector® Fluid polish is a concentrate for sole use with the Vector® System of Dürr Dental SE, that is automatically diluted with water to the correct usage concentration.

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Follow the instructions for use of the Vector® System precisely. Shake before use.

### Warning:

Do not swallow. Avoid contact with the eyes. If there is contact with the eyes, rinse thoroughly with water. Do not remove the cap from the pouch. All serious events that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state.

### Precautions:

The patient and operator need to wear suitable eye protection. The operator additionally needs to wear mouth and nose protection and suitable protective gloves.

### Storage and shelf life:

Store in enclosed, well-ventilated rooms away from moisture and bright light between 5°C and 25°C. When the storage conditions are observed, the shelf life is 18 months in the sealed container and 4 weeks after opening.

### Date of opening:

enter on the back of the pouch (white area).

### Disposal:

Dispose of in accordance with the local official regulations.

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DÜRR DENTAL ITALIA S.r.l. Via Dell'Artigianato n. 8, 20053 Muggiò (MI), Italy ☎ +39 0395970300